

# proteX<sup>®</sup> ultra

disinfectant wipes

## One-Step<sup>4</sup>/Disinfectant<sup>6</sup>/Sanitizer<sup>7</sup>

Premoistened disinfecting wipes

- Kills 99.9% of bacteria in just 15 seconds • Effective in the presence of 5% serum contamination
- Hospital Disinfectant • Effective against SARS-Related Coronavirus 2<sup>4</sup> (SARS-CoV-2) in 4 minutes

### Protex Ultra Disinfectant Wipes Are Effective Against:

- Staphylococcus aureus-Multi-Drug Resistant
- Staphylococcus aureus-Vancomycin Intermediate Resistant-VISA
- Pseudomonas aeruginosa • Burkholderia cepacia
- \*Hepatitis B Virus [HBV] • \*Hepatitis C Virus [HCV]
- \*HIV-1 [AIDS Virus] • \*SARS-Related Coronavirus 2 (SARS-CoV-2) (cause of COVID-19)

### KEEP OUT OF REACH OF CHILDREN

### CAUTION See back panel for Precautionary Statements and First Aid

**Use Protex Ultra Disinfectant Wipes in:** Ambulances • Dental Offices • Emergency Vehicles • Exercise Facilities • Health Clubs • Hospitals • Locker Rooms • Medical Offices

**Use Protex Ultra Disinfectant Wipes on washable hard, non-porous surfaces of:** Non-critical medical device surfaces: Defibrillators • Resuscitators • Stethoscopes • Ultrasound transducers and equipment

Other hard, non-porous surfaces made of: Finished woodwork • Metal • Plastic • Stainless steel • Upholstery – vinyl and plastic

To clean and disinfect hard non-porous surfaces of gym equipment, exercise machines, wrestling mats and gymnastic equipment.

#### Protex Ultra Disinfectant Wipes kills: Bacteria

Acinetobacter baumannii, Corynebacterium ammoniagenes, Campylobacter jejuni, Staphylococcus aureus-Community Associated Methicillin-Resistant [CA-MRSA] [NRS384] [USA300], Enterobacter aerogenes, Enterococcus faecalis-Vancomycin Resistant [VRE], ESBL Escherichia coli, Klebsiella pneumoniae, Klebsiella pneumoniae (NDM-1), Legionella pneumophila, Pseudomonas aeruginosa, Burkholderia cepacia, Salmonella enterica, Salmonella schottmuelleri, Salmonella typhi, Serratia marcescens, Shigella dysenteriae, Staphylococcus aureus-Methicillin-Resistant [MRSA], Staphylococcus aureus-Multi-Drug Resistant, Staphylococcus aureus-Vancomycin Intermediate Resistant – [VISA], Streptococcus pyogenes, Vibrio cholerae

#### Viruses

\*Hepatitis B Virus [HBV], \*Hepatitis C Virus [HCV], \*Herpes Simplex Virus Type 1, \*Herpes Simplex Virus Type 2, \*HIV-1 [AIDS Virus], \*Human Coronavirus, \*Influenza Type A/Brazil Flu, \*Respiratory Syncytial Virus, \*SARS-Related Coronavirus 2 (SARS-CoV-2)(cause of COVID-19), \*Vaccinia Virus

**To Open Package:** Slowly peel back the label. Remove wipe as needed. To retain moisture, reseal by firmly pressing seal back on the pouch.

#### DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. Do not use as a diaper wipe or for personal cleansing. This is not a baby wipe! Do not flush in toilet.

For plastic and painted surfaces, spot test on an inconspicuous area before use. Will not harm most surfaces: including acrylic, sealed fiberglass and vinyl.

#### TO CLEAN AND DISINFECT/VIRUCIDAL\*

**DIRECTIONS:** Wipe hard non-porous surface with wipe until surface is visibly wet. Allow surface to remain wet for 4 minutes. Permit surface to air dry. If surfaces are visibly dirty, clean first with another wipe before disinfecting. Discard used wipe in trash. Do not flush in toilet.

**\*KILLS HIV AND HBV AND HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 [HIV-1] or Hepatitis B Virus [HBV] or Hepatitis C Virus [HCV].

#### SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OR HBV OR HCV ON SURFACES / OBJECTS SOILED WITH BLOOD / BODY FLUIDS.

**Personal protection:** Clean-up must always be done wearing protective gloves, gowns, masks and eye protection.

**Cleaning procedure:** Blood and other body fluids containing HIV or HBV or HCV must be thoroughly cleaned from surfaces and objects before application of **Protex Ultra Disinfectant Wipes**. **Protex Ultra Disinfectant Wipes** can be used for this purpose.

**Contact time:** Leave surface wet for 1 minute (60 seconds) for HIV-1. Use appropriate contact time as listed on label for all other viruses and bacteria claimed.

#### Active Ingredients:

Octyl decyl dimethyl ammonium chloride.....	0.0909%
Dioctyl dimethyl ammonium chloride.....	0.0364%
Didecyl dimethyl ammonium chloride.....	0.0545%
Alkyl (C <sub>14</sub> , 50%; C <sub>12</sub> , 40%; C <sub>16</sub> , 10%) dimethyl benzyl ammonium chloride.....	0.1212%
Other Ingredients: .....	99.6970%
Total.....	100.0000% <sup>1</sup>

<sup>1</sup>Includes weight of absorbent wipe

**Disposal of infectious material:** Blood, body fluids, cleaning materials and clothing must be disposed of according to local regulations for infectious waste disposal.

**SANITIZING:** To sanitize hard, non-porous, non-food contact surfaces; wipe surface using enough wipes to allow treated surface to remain wet for 15 seconds for Staphylococcus aureus, Klebsiella pneumoniae, and Campylobacter jejuni. Let surface dry. For visibly soiled surfaces, clean excess dirt first.

This product is not to be used as a terminal sterilant / high level disinfectant on any surface or instrument that is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

#### STORAGE AND DISPOSAL

Store in original container in areas inaccessible to children. Keep securely closed. Nonrefillable container. Do not reuse or refill this container. Discard empty container in trash or recycle (if available).

#### PRECAUTIONARY STATEMENTS HAZARDS TO HUMANS AND DOMESTIC ANIMALS

**Caution.** Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

#### FIRST AID

**IF IN EYES:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling poison control center or doctor or going for treatment.

<sup>4</sup>On hard, non-porous, non-food surfaces

<sup>6</sup>When used according to the disinfection directions

<sup>7</sup>When used according to the sanitization directions

MANUFACTURED FOR:



Parker Laboratories Inc.  
286 Eldridge Road  
Fairfield, NJ 07004  
parkerlabs.com

**Net Contents:** 80 premoistened disinfecting wipes, 7 in. X 10 in.  
**Minimum Wt.:** 1.45 lbs.

EPA Registration No. 6836-340-82613  
EPA Establishment No. 95598-AZ-1

**Questions? Comments?**  
**Call 1-800-631-8888**